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Instructions for Use**BlephaCura®**

liposomal suspension for the relief of inflammation of the eyelid margin and for daily care of the eyelid margin

Dear Users!

Please read these Instructions for Use carefully, because they contain important information about what you should pay attention to when using this medical device. If you have any questions, please contact your doctor or pharmacist.

Ingredients

Soy lecithin, vitamin A (palmitate), vitamin E, ethanol, sodium chloride, phenoxyethanol, aqua purificata

Therapeutic Indications

BlephaCura® serves to relieve and treat inflammation of the eyelid margin; it is also used in the daily care and hygiene of the eyelid margin.

BlephaCura® supports and promotes the natural healing process in inflammation and irritation of the eyelid margins.

Application

It is recommended that **BlephaCura®** be used twice daily in the morning and evening.

BlephaCura® should be applied with fluff-free (cotton) pads that are suitable for eyelid care. To avoid contamination, the dropper should come into contact with any surfaces, especially not the cotton pad. Please drop **BlephaCura®** onto the cotton pad at a small distance.

During daily care and hygiene of the eyelid margin with **BlephaCura®**, all crusts and deposits should be thoroughly removed. It is recommended that application be undertaken with previously cleaned hands.

Tips for correct use:

- Moisten a cotton pad with **BlephaCura®**
- Then moisten the eyelids, especially the eyelid margins and the base of the eyelashes with a saturated cotton pad and gently massage the eyelid margin along the upper and lower edge of the eyelid
- Carefully remove any crusts and deposits that are present. To treat the eyelid margins, it is recommended that the lower eyelid gently be pulled downward with one finger of the free hand or respectively the upper eyelid be gently pulled upward, so that the eyelid margin can be easily reached. If necessary, replace the used cotton pad with a new one.

Discard the cotton pad used after treating each eyelid; do not use the same cotton pad for both eyelids.

Do not place **BlephaCura®** directly into the eyes. Do not treat the inside of the eyelids or the eye surface with the saturated cotton pad.

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The frequency and duration of use of **BlephaCura®** is not limited on principle.

Contraindications and Side Effects

None known.

Notices

BlephaCura® should not be used in the presence of hypersensitivity to one of the ingredients.

Background Information

An inflammation of the eyelid margin (blepharitis) is usually a chronic disease whose treatment and course can be associated with disappointing setbacks and recurrences and, therefore, requires careful, daily care.

As needed, it can prove helpful to place warm compresses (37–38 °C) on the closed eyelids for about 15 minutes before treatment with **BlephaCura®**. This facilitates the liquefaction of any existing mucous clots and secretions from the sebaceous glands and their subsequent problem-free removal.

Liposomes rapidly penetrate the skin, supply it with water, phospholipids and essential fatty acids such as oleic acid, linoleic acid, linolenic acid and vitamin E. They possess pain-relieving and antiphlogistic properties that support the rapid abatement of any inflammations. Liposomes promote and support wound healing and care for the irritated and inflamed skin.

Shelf-life and Storage

The expiration date of this package is printed on the outside of the folding carton and on the label. Do not use after the expiration date has elapsed.

After opening, the contents can be used for 4 weeks.

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Optima Pharmazeutische GmbH