Rosidal® K

CO

Elko®-Rosidal® K

2 Rosidal® K, 10 cm

Durelast®

66% CO, 34% PA

Durelast® Combi

2 Durelast®, 8 cm und 10 cm

Rosidal® haft

100% CO

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Short stretch bandages for compression, support and relief
bandaging
Instructions for Use
Indications:

• **Compression bandaging** for phlebological and lymphological
indications, e.g. varices, insufficiency of the perforator veins, thrombo-
phlebitis, phlebothrombosis, post-thrombotic syndrome, chronic venous
insufficiency stages I to III, in cases of oedemas (phleboedema,
lymphoedema, lipoedema, posttraumatic, postoperative, during
pregnancy), after varicose vein procedures.

• **Support and relief bandaging** for the prophylaxis and treatment
of contusions, distorsions, tendon injuries and luxations.

Contraindications:
Advanced peripheral occlusive arterial disease, decompensated heart
insufficiency, septic phlebitis, phlegmasia coerulea dolens.

Important notes:
Rosidal® haft must be unrolled about 20 cm prior to application.

Elko®-Rosidal® K - For compression bandaging in double bandage
techniques (fig. A), the second Rosidal® bandage must be applied over
the first one in the opposite direction using the same technique. Start
either at the ankle joint or at the base of the toes (see fig. A10 and A11).
Durelast® Combi may be applied according to Sigg's compression
bandaging technique (fig. B). To start bandaging, use the 8 cm wide
Durelast®. From the calf onwards, the 10 cm bandage is applied.

General notes:

1. Ideally apply the bandage when the swelling is down, preferably
after the patient has just got up in the morning or after having rested
the leg in a raised position for some time.
2. Select a bandage which is equal in width to the diameter of the limb
to be treated. Bandages, 8, 10 or 12 cm wide, are most commonly
used.
3. Unless otherwise indicated start bandaging on the foot which
should form an angle of 90° with the leg.
4. Apply the bandage at consistent pressure to ensure graduated
compression from the foot to the knee.
5. Apply the bandage holding the roll close to the limb and pointing
outward.
6. Ask the clinician to indicate the particular bandaging technique he
or she wishes to be applied.
7. Inexpert bandaging may lead to cutaneous gangrene and nervous
pressure injuries.

Special washing instructions for Rosidal® haft:
Before washing, position in zig-zag winds measuring approx. 20 cm
and squeeze together.

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Lohmann & Rauscher International GmbH & Co. KG, D-56579 Rengsdorf