23308364

Coloplast Purilon, Biatain, SeaSorb and Comfeel are registered trademarks of Coloplast A/S or related companies.
© 2007-06. All rights reserved.
Manufacturer: Coloplast A/S, 3050 Humlebæk, Denmark. Réf. 351.

Purilon Gel

English, Deutsch, Français, Italiano,
Nederlands, Español, Português,
Dansk, Norsk, Svenska, Suomi, Polska

Purilon

Coloplast A/S
Holtedam 1
3050 Humlebæk
Denmark

www.coloplast.com

English

Purilon Gel

Remove the label from accordion pack by pulling the corner as indicated.

Clean the nozzle below the snap-off tip with a suitable antiseptic swab.

Remove snap-off tip.

Gently press the base of accordion pack to apply the **Purilon Gel** in the required amount.

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Indications

Purilon Gel is primarily intended for the treatment of necrotic and sloughy wounds, leg ulcers, pressure ulcers and non-infected diabetic foot ulcers.

It may also be used on 1st and 2nd degree burns.

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Precautions

Cleanse the wound with physiological saline or tap water. Gently dry the skin around the wound.

To optimise the use of **Purilon Gel**, apply according to illustrations.

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Cover with a secondary dressing. For wounds with medium exuding wounds, choose hydrocolloid as a secondary dressing, e.g. **Comfeel Plus**. For medium to highly exuding wounds, choose a highly absorbent or highly permeable dressing to cover, e.g. **Biatain Dressings**.

To optimise cleansing of necrotic and sloughy wounds, it is recommended to change **Purilon Gel** at least every 3rd day, due to the larger amount of exudate. For clean wounds **Purilon Gel** should be changed depending on the amount of exudate.

The gel can be removed from the wound by rinsing with physiological saline or tap water.

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ound.
to optimise the use of **Purilon** Gel, apply according to illustrations.

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Cover with a secondary dressing. For low to medium exuding wounds, choose hydrocolloid as a secondary dressing, e.g. **Comfeel** Plus. For medium to highly exuding wounds, choose a highly absorbent or highly permeable dressing to cover, e.g. **Biatain** Dressings.

To optimise cleansing of necrotic and heavily exuding wounds, it is recommended to change **Purilon** Gel at least every 3rd day, due to the larger amount of exudate. For clean wounds **Purilon** Gel should be changed depending on the amount of exudate.

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Precautions

Do not be used on patients with local and systemic infections under the discretion of a healthcare professional.

Wounds which are solely or mainly caused by an arterial insufficiency or diabetic wounds (primarily lower leg and foot) should be inspected by a healthcare professional, dependent on the clinical condition.

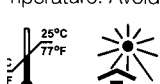
In case of allergic reactions, please contact Coloplast for information on components.

Due to the effective debridement of necrotic tissue, a wound being treated with **Purilon** Gel may appear larger at first dressing change, however, this is a natural step in the healing process.

For external use only.

Storage

Purilon Gel should be stored at room temperature. Avoid direct sunlight.

Sterilisation

If packaging is broken do not use or re-use product.

Do not reuse.

Work Order (WO)

The production number identifying the packaging of an order for a specific country. The LOT-number relates to production run).

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23308364

Coloplast Purilon, Biatain, SeaSorb and Comfeel are registered trademarks of Coloplast A/S or related companies.
© 2007-05. All rights reserved.
Manufacturer: Coloplast A/S, 3050 Humlebæk, Denmark. Réf. 351.

Coloplast A/S
Holtedam 1
3050 Humlebæk
Denmark

www.coloplast.com

Purilon Gel

English, Deutsch, Français, Italiano,
Nederlands, Español, Português,
Dansk, Norsk, Svenska, Suomi, Polska

Purilon

English

Purilon Gel

1. Remove the label from accordion pack by pulling the corner as indicated.
2. Clean the nozzle below the snap-off tip with a suitable antiseptic swab.
3. Remove snap-off tip.
4. Gently press the base of accordion pack to apply the **Purilon** Gel in the required amount.

Description

Purilon Gel is a clear and amorphous hydrogel for effective autolytic debridement of necrotic tissue.

Indications

Purilon Gel is primarily intended for the treatment of necrotic and sloughy wounds, e.g. leg ulcers, pressure ulcers and non-infected diabetic foot ulcers.

May also be used on 1st and 2nd degree burns.

The gel may be used throughout the healing process to provide a moist wound healing environment in all types of wounds, except those mentioned under precautions.

Directions

Rinse the wound with physiological saline or tap water. Gently dry the skin around the wound.

To optimise the use of **Purilon** Gel, apply according to illustrations.

The wound should not be filled to a higher level than the surrounding skin.

Cover with a secondary dressing. For low to medium exuding wounds, choose a hydrocolloid as a secondary dressing, e.g. **Comfeel** Plus. For medium to highly exuding wounds, choose a highly absorbent or highly permeable dressing to cover, e.g. **Biatain** Dressings.

To optimise cleansing of necrotic and sloughy wounds, it is recommended to change **Purilon** Gel at least every 3rd day, due to the larger amount of exudate. In clean wounds **Purilon** Gel should be changed depending on the amount of exudate.

The gel can be removed from the wound by rinsing with physiological saline or tap

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Purilon Gel, apply
filled to a higher
g skin.

dressing. For
wounds, choose
secondary dressing,
medium to
choose a highly
neable dressing to
sings.

necrotic and
recommended to
fast every 3rd
amount of exudate.
Gel should be
the amount of

from the wound
ical saline or tap

s with local and
for the discretion of a

/ or mainly caused
y or diabetic
(leg and foot) should
care professional,
condition.

ns, please contact
on components.
idement of necrotic
eated with **Purilon**
: first dressing
a natural step in

ored at room
t sunlight.

not use or re-

ifying the
a specific country
roduction run).

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