EULATIN® NH Ointment

Medical device to relieve bowel movement when suffering from haemorrhoids

INDICATIONS FOR USE

1. Therapeutic purpose

Eulatin® NH Ointment achieves a physical relief of bowel movement by lubricating the rectum, the anus and a protection of the imperforate anal mucosa in case of haemorrhoids. The product improves the sliding properties of stool and protects the mucosa from laceration and itching.

2. Therapeutic indications

Eulatin® NH Ointment is ready to use for internal and external haemorrhoids in the end region of the rectum and of the anus for the physical protection of the imperforate anal mucosa in adults and for the prevention of pathological itching.

The products can be applied in professional care of patients, the handicapped and elderly, personal and in domestic use.

3. Undesirable effects

Undesirable effects are not known for this product when applied as prescribed.

4. Histocompatibility

No data is available that suggests there may be a histocompatibility of the tissue.

5. Contraindications

Eulatin® NH Ointment may not be applied when allergies to one the ingredients are known.

Attention:

The product may not be applied during pregnancy or lactation.

6. General special warnings and special precautions for use

The product should not be applied when blood is discharged or blood is suspected in stool and there is simultaneous occurrence of pain and fever.

The product is solely designed for external application to the treatment area and may not enter the body by ingestion.

Attention:

Keep out of the reach and sight of children.

Attention:

When using *Eulatin® NH Ointment* in the area of the anus and at the same time using condoms of latex, the tensile strength may be reduced and thus lead to reduced protection from the condom due to the excipients petrolatum and paraffin.

7. Product properties

Haemorrhoids cause discomfort which, depending on the severity of the disorder may lead to itching, bleeding, inflammation or thrombosis. Severe pain is experienced particularly in bowel movement. As a reflex to the severe pain experienced in bowel movement, unconditioned constipation occurs, even more pressing strain is required for bowel movement which leads to a further amplification of existing haemorrhoids.

Eulatin® NH Ointment lubricates the anal passage and gives protection against haemorrhoids. The stool can slide more easily through the pre-lubricated anal passage, bowel movement therefore requires little pressure which avoids

painful pressing. In addition, the anal mucosa is optimally protected against injury by the covering properties of the product.

8. Product application

The anal areas should be cleaned carefully before each application of *Eulatin® NH Ointment*.

In the case of external haemorrhoids, use a finger to take the product from the tube and apply directly.

For application to the rectum (for internal haemorrhoids) screw the enclosed applicator (accessory) to the tube and lubricate the tip with the product. Then insert the applicator very carefully into the anus and press the product slowly from the tube. Clean the outside of the applicator after use.

It is advisable to cover the application area with cellulose or something similar to protect clothing. Apply *Eulatin® NH Ointment* by inserting applicator into the rectum two to three times daily, or before bowel movement respectively.

9. Physical product data

Composition:

Eulatin® NH Ointment contains:

Lanolin, lanolin alcohol ointment, white petrolatum, purified water, medium-chain triglycerides, hard paraffin, alkaline bismuth gallate, fluid extract of hamamelis bark, tannic acid, medicinal yeast, highly-disperse silicon dioxide,

Appearance, odour: yellowish ointment, odourless

Shelf-life: 3 years

The minimum shelf-life is printed on the tube and the folding box of the product.

Special precautions for storage:

Store *Eulatin® NH Ointment* at room temperature but not above 25°C.

10. Delivery specifications

The outer packaging of the product contains:

Tube with 30 g or 60 g contents

Accessory: Applicator for internal application in the rectum.

11. Manufacturer / Distributor

Manufacturer and distributor of the medical product:

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RIEMSER Arzneimittel AG