Medihoney™ Antibacterial Honey is a standardised antibacterial honey, predominantly *Leptospermum sp.*, selected for its unique wound cleaning and antibacterial barrier properties.

Medihoney™ Antibacterial Medical Honey™ is a topical preparation which contains 100% Medihoney™ Antibacterial Honey and can be used for both chronic and acute wound care. It is proven to be clinically effective for:

- providing a moist wound healing environment
- protecting the wound by creating a barrier against wound pathogens including antibiotic resistant strains
- providing a cleaner wound
- rapidly removing malodour
- reducing the risk of infection from bacteria

Medihoney™ Antibacterial Medical Honey™ is a non-adherent natural dressing and therefore reduces trauma and pain at dressing changes.

Indications:

- Sinus wounds
- Deep wounds
- Infected wounds
- Sloughy wounds
- Necrotic wounds
- Malodorous wounds
- Surgical wounds
- Superficial wounds such as cuts, scratches, abrasions
- Superficial burns

Serious wounds, such as deep, sinus, necrotic, diabetic, infected wounds, sloughy, malodorous or surgical wounds, should be managed under the supervision of a health care professional.

Directions for use:

1. Protect wound edges with a barrier cream
2. Medihoney™ Antibacterial Medical Honey™ can be directly applied to the base of the wound. Avoid direct contact between the wound and the nozzle of the tube.
3. Medihoney™ Antibacterial Medical Honey™ can be directly applied into deep wound cavities or soaked into a suitable dressing.
4. Ensure that Medihoney™ Antibacterial Medical Honey™ is in full contact with the base of the wound – approximately 3mm thickness (1/4 inch). Apply more if wound fluid is present.
5. A suitable non-adherent sterile pad or dressing should be applied to cover the Medihoney™ Antibacterial Medical Honey.
6. The dressing should be sufficiently absorbent to manage wound fluid (exudate)
7. Medihoney™ Antibacterial Medical Honey™ can be left on the wound for up to 7 days dependent on the amount of fluid present. The dressing should be changed when it is saturated with wound fluid or if the cover dressing is leaking.

Contraindications

- Do not use on individuals who have had an allergic reaction to honey.

Observations

- The wound may appear larger in size and depth as dead tissue is cleaned away. This increase should be followed by an improved appearance of the wound.

Precautions

- For any experience of a reaction other than a short lasting stinging sensation, remove the dressing and wash the affected area.
- To be used with caution and close observation on heavily exuding wounds.
- Not recommended for use in body cavities.
- In the event of a slow or non-responding wound please seek medical advice.

For Single Patient Use

Product is only sterile until seal is broken. Do not use if tube is damaged.

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